



CoproELISA™ *Entamoeba*

Enzyme-Linked Immunosorbent Assay (ELISA)
For the detection of *Entamoeba* antigens
in human feces

Instruction Manual

Test kit for 96 determinations
Catalog Number: 754-01

For *In Vitro* Diagnostic Use
For professional use only
Store at 2-8°C. **Do Not Freeze**



Savyon® Diagnostics Ltd.

3 Habosem St. Ashdod 77610

ISRAEL

Tel. +972.8.8562920

Fax: +972.8.8523176

E-mail: support@savyondiagnosics.com

Intended Use

Savyon's CoproELISA *Entamoeba* test is an Enzyme-Linked Immunosorbent Assay (ELISA) for detection of *Entamoeba* antigens in unpreserved human fecal specimens collected from patients with gastrointestinal symptoms. The test can be used for fecal specimens submitted for routine clinical testing from adults or children.

For *In-Vitro* Diagnostic Use

Introduction

Entamoeba histolytica/dispar (*E. histolytica/dispar*) are intestinal protozoan parasites that are transferred via the faecal-oral route and infect up to 10 % of the human population (1). *Entamoeba* infections are most common in the developing world and are associated with poor sanitation where the *Entamoeba* cysts are transmitted from contaminated food and water. After ingestion trophozoites the *Entamoeba* vegetative form are released from their cyst capsules into the small intestine. Trophozoites which survive strictly in the host intestine and fresh stool migrate to the large intestine where they live and multiply by binary fission. Trophozoites of *E. dispar* are noninvasive and reside in the intestine while those of *E. histolytica* can invade the intestinal mucosa causing human amoebiasis (1 and 2). Symptoms of amoebiasis can range from mild diarrhea to dysentery with blood and mucus in the stool. While most *E. histolytica* infected people remain asymptomatic, without proper treatment, this disease can lead to severe complications, including abscess of vital organs, most commonly in the liver, and in some cases in the lungs, and the brain (3). It is estimated that *E. histolytica*, represents

10% of human *Entamoeba* infections which cause the death of between 40,000 to 100,000 people every year (4). It is, therefore, important to diagnose the infection and provide appropriate treatment. The most common treatment for amoebiasis is the administration of antibiotics, such as metronidazole and tinidazole.

Principle of the Test

- Plates are coated with specific polyclonal antibodies directed against *Entamoeba* antigens.
- Fecal sample to be tested is diluted in stool diluent and incubated with the pre-coated plate. In this step *Entamoeba* antigens are bound to the immobilized antibodies.
- Non-specific antigens are removed by washing.
- Anti- *Entamoeba* monoclonal antibody conjugated to horseradish peroxidase (HRP) is added and incubated. In this step the HRP-conjugate is bound to the pre-bound antigen-antibody complex.
- Unbound conjugate is removed by washing.
- Upon the addition of TMB-substrate, the substrate is hydrolyzed by the peroxidase, yielding a blue solution of the reduced substrate.
- Upon the addition of the stop solution, the blue color turns yellow and should be read by an ELISA reader at a wavelength of 450/620 nm.
- The absorbance is proportional to the number of *Entamoeba* cells present in the sample.

Summary of Procedure Manual/Automation*



*Automation Procedure:

- 50 minutes sample incubation
- Wash cycles volume: 500 µl /well
- 10 minutes substrate incubation

Kit contents for Automation use

Test Kit for 96 determinations:

1. **Microtiter plate coated with anti – *Entamoeba* polyclonal antibodies:**

96 break-apart wells (8x12) coated with anti-*Entamoeba* polyclonal antibodies, packed in an aluminum pouch containing a desiccant card.

1 plate

2. **Concentrated Wash Buffer (20x):** A PBS -Tween buffer.

1 bottle, 100 ml

3. **Stool Diluent (Blue):** A ready-to-use buffer solution. Contains less than 0.05% Proclin as preservative. The Diluent is also to be used as the negative control solution (see TEST PROCEDURE)

2 bottles, 50 ml

4. **HRP-Conjugate (Green):** A ready-to-use solution containing Horseradish peroxidase (HRP) conjugated anti-*Entamoeba* monoclonal antibody. Contains less than 0.05% Proclin as preservative.

1 bottle, 16ml

5. **Positive Control:** A ready to use solution containing *Entamoeba* antigen. Contains less than 0.05% Proclin as preservative.

1 vial, 2.5 ml

6. **TMB-Substrate:** A ready to use solution contains 3,3',5,5' tetramethylbenzidine as a chromogen and peroxide as a substrate.

1 1 bottle, 16 ml

7. **Stop Solution:** A ready to use solution. Contains 1M H₂SO₄.

1 bottle, 16 ml

8. **Disposable plastic pipettes:** **100 pc**

9. **Plate cover:** **1 unit**

10. **Instruction Manual:** **1 unit**

Materials Required But Not Supplied:

1. Clean test tubes for dilution of patients' stool.
2. Adjustable micropipettes, or multichannel pipettes (50-200 and 200-1000µl ranges) and disposable tips.
3. Disposable plastic/wooden collectors or teaspoons.
4. One-liter volumetric flask.
5. One 50 ml volumetric cylinder.
6. Wash bottle.
7. Absorbent paper.
8. Vortex mixer.
9. A 37°C water bath with a lid, or a moisture chamber placed in a 37°C incubator.
10. ELISA-reader equipped with 450/620 nm filters.
11. Distilled or double de-ionized water.

Warnings and Precautions

1. Reagents should be brought to room temperature before use.
2. When handling assay wells, avoid scratching the bottom of the wells because this may result in elevated absorbance readings.
3. Stool samples, microassay wells, micropipette tips and disposable stool collectors and tubes should be handled and disposed of as potentially biohazards after use. Wear gloves when doing the test.
4. **Unused microassay wells must be replaced in the re-sealable pouch with the desiccant to protect them from moisture.**

5. TMB-Substrate solution is an irritant material to skin and mucous membranes. Avoid direct contact.
6. Diluted sulfuric acid (1M H₂SO₄) is an irritant agent for the eyes and skin. In case of contact with eyes, immediately flush area with water and consult a physician).

Storage and Shelf-Life of Reagents

1. The expiration date of the kit is given on the label. Expiration dates for each component are listed on individual labels. The kit should be stored between 2° and 8°C and should be returned to the refrigerator as soon as possible after use. Exposure of originally stoppered or sealed components to ambient temperature for a few hours will not cause damage to the reagents. **DO NOT FREEZE!**
2. Unused strips must be resealed in the aluminum pouch with the desiccant card, by rolling the open end and sealing tightly with tape over the entire length of the opening.

Stool Collection

1. Standard collection and handling procedures used in-house for fecal specimens or culture are appropriate.
2. **Preserved stool: The test is not compatible with specimens that were fixed in 10% formalin or in Sodium Acetate Formalin (SAF). The test is also not compatible with stool specimens fixed in Polyvinyl Alcohol (PVA).**
3. **Unpreserved specimens:** Unpreserved specimens should be stored between 2° and 8°C and tested within 48 hours after collection. If testing cannot be performed within 48 hours, store samples at -20°C, or lower.
4. Freezing and thawing of the specimen, especially multiple times, may result in loss of activity due to degradation or proteolysis of the antigens.

Test Procedure for manual use

A. Preparation of Reagents

1. Bring all components and clinical specimens to be tested to room temperature. Determine the total number of specimens to be tested. In addition to the specimens, the following must be included in each test: one well of Negative Control (Use Stool Diluent for this purpose) and one well of Positive Control.
2. Withdraw the microtiter plate from its aluminum pouch by cutting one end near the seal. Leave the required number of strips (according to the number of specimens to be tested) in the 96 well frame.
3. Dilute the Concentrated Wash Buffer 1/20 with double-deionized or distilled water. For example, in order to prepare one liter of Wash Buffer, add 50 ml of the Concentrated Wash Buffer to 950 ml of double-deionized or distilled water.

B. Sample Processing

4. Set up one dilution tube for each specimen to be tested. 1.5 mL Eppendorf tubes are recommended for this purpose. Add 400 µL Stool Diluent to each tube. Label the tube.
5. **Formed samples:** Use a wooden applicator stick or a disposable teaspoon to transfer the fecal specimen to the tube. Transfer approximately 0.1 to 0.15 g of specimen (about the size of a small pea) to the stool Diluent. Mix the collector in the *Stool Diluent* to remove as much

sample as possible and squeeze the collector against the side of the tube to express any residual liquid.

Liquid samples: transfer 150 µL of specimen to the tube. Make sure the liquid specimens are evenly suspended.

6. **Thoroughly mix (vortex) the fecal specimen to ensure adequate sampling.**
7. Let the tube stand for at least 10 minutes but not more than 30 minutes until large particulate matter is precipitated (decantation). Use upper liquid phase for testing. **DO NOT USE CENTRIFUGE FOR THIS PURPOSE**

C. Incubation of stool samples and controls

8. Thoroughly vortex the positive control tube for 10 seconds and immediately pipette 100 µL of Positive control and 100µL of Negative Control (i.e., Stool Diluent) into separate wells of the test strip.
9. Dispense 100 µL of diluted stool samples into separate wells of the test strip using the provided disposable pipettes (the lowest mark on the pipette).
10. Cover the strips with a plate cover and incubate for 1h at 37°C in a moisture chamber.
11. **Washing step:** Discard the liquid content of the wells. Fill each well with Wash Buffer up to the end of the well (300 µL). Repeat this step 4 times to a total of **FIVE** times. Automatic washing machine can be used.
12. Dry the strips and frame by gently tapping them over clean absorbent paper.

D. Incubation with Conjugate

13. Dispense 100µL of ready-to-use conjugate into each well.
14. Cover the strips with a plate cover and incubate for 1h at 37°C in a moisture chamber.
15. Discard the liquid content and wash **FIVE** times as described in steps 11-12-.

E. Incubation with TMB Substrate

16. Dispense 100 µL of TMB-Substrate into each well, cover the strips with a plate cover and incubate at room temperature for **15 minutes**.
17. Stop the reaction by adding 100µL of Stop Solution (1M H₂SO₄) into each well.

F. Determination of Results

18. Determine the absorbance at 450/620 nm and record the results. Determination should not exceed 10 minutes following stopping of the chromogenic reaction.

Note: Any air bubbles should be removed before reading. The bottom of the ELISA plate should be carefully wiped

Test Procedure for automation use

A. Preparation of Reagents

1. Bring all components and clinical specimens to be tested to room temperature. Determine the total number of specimens to be tested. In addition to the specimens, the following must be included in each test: one well of Negative Control (Use Stool Diluent for this purpose) and one well of Positive Control
2. Withdraw the microtiter plate from its aluminum pouch by cutting one end near the seal. Leave the required number of strips (according to the number of specimens to be tested) in the 96 well frame.

3. Dilute the Concentrated Wash Buffer 1/20 with double-deionized or distilled water. For example, in order to prepare one liter of Wash Buffer, add 50 ml of the Concentrated Wash Buffer to 950 ml of double-deionized or distilled water.

B. Sample Processing

4. Set up one sample's dilution tube for each specimen to be tested (use sample's tubes compatible with the available automation equipment). Add 800 µL Stool Diluent to each sample's tube. Label the tube.
5. **Formed samples:** Use a wooden collector or a disposable teaspoon to add the fecal specimen to the sample's tube. Transfer approximately 0.2 to 0.3 g of specimen (about the size of 2 small peas) to the sample's tube. Mix the collector in the *Stool Diluent* to remove as much sample as possible and squeeze the collector against the side of the tube to extract any residual liquid.
Liquid samples: transfer 300µL of specimen to the tube. Make sure the liquid specimens are evenly suspended.
6. **Thoroughly mix (vortex) the fecal specimen to ensure adequate sampling.**
7. Let the tube stand for at least 10 minutes. Centrifuge the tubes at 1000 g for 30 sec. Ensure that the formed supernatant does not contain large particulate material.
8. Transfer the sample's tubes to the corresponding rack at the automation machine.

C. Incubation of stool samples and controls

9. Pipette 100 µL of Positive control and 100µL of Negative Control (i.e., Stool Diluent) into separate wells of the test strip.
10. Dispense 100 µL of diluted stool samples into separate wells of the test strip.
11. Incubate the plate at 37°C for **50** minutes.
12. Perform 5 X **500 µl** wash cycles using the pre-diluted Wash Buffer.
13. Perform 2 aspirate cycles with aspirate sweep.

D. Incubation with Conjugate

14. Dispense 100µL of ready-to-use conjugate into each well.
15. Incubate for 1h at 37°C.
16. Repeat washing cycles as described in steps 7-8.

E. Incubation with TMB Substrate

17. Dispense 100 µL of TMB-Substrate into each well. Incubate at room temperature for **10 minutes**.
18. Stop the reaction by adding 100µL of Stop Solution (1M H₂SO₄) into each well.

Test Validation

For the test to be valid the following criteria must be met. If these criteria are not met the test should be considered invalid and should be repeated.

1. **Positive Control:** The absorbance value should be ≥ 1.0 at 450/620 nm.
2. **Negative Control:** The absorbance value should be ≤ 0.25 at 450/620 nm.

Determination of Cut-Off Value

1. The cut-off value (COV) is **0.15**

Interpretation of Results

Absorbance (450/620nm)	Results
O.D < 0.15	Negative: no detectable <i>Entamoeba</i> antigen
O.D ≥ 0.15	Positive: relevant levels of <i>Entamoeba</i> antigen

Test Limitations

1. **The test is not compatible with stool specimens fixed in Formalin, SAF or Polyvinyl Alcohol (PVA).**
2. Positive result does not exclude the presence of other etiologies. It is therefore advised to take into account all clinical and laboratory data before making final diagnosis and decide upon appropriate patient management.

Performance Characteristics of the Test

Study 1: was performed in several medical centers (Israel, Holland, and Nicaragua) across the globe, a total of 126 stool specimens were evaluated by CoproELISA™ *Entamoeba* test. The presence of gastrointestinal parasites in these specimens was pre-determined by microscopy (and secondly confirmed by PCR). The results of this evaluation are shown in Table 1:

Table 1:

CoproELISA™ Entamoeba	Microscopy	
	Positive	Negative
Positive	55	3
Negative	7	61

Sensitivity: 87.7% Specificity: 95.3%
PPV: 94.8% NPV: 89.7%

Study 2: In a second study performed in-house a total of 84 pre-defined stool specimens by microscopy were evaluated by CoproELISA™ *Entamoeba* test. The results shown below were compared with available commercial reference ELISA kit.

Table 2:

CoproELISA™ Entamoeba	Competitor ELISA	
	Positive	Negative
Positive	40	2
Negative	0	42

Sensitivity: 100% Specificity: 95.4%
PPV: 95.2% NPV: 100%

Cross Reactivity and Interference by Mixed infections

The CoproELISA™ *Entamoeba* test was evaluated using stool specimens defined as positive for various gastrointestinal pathogens. No cross-reactivity of interference by mixed infection with any of the pathogens listed below:

Blastocystis, *C. difficile*, *Cryptosporidium*, *E. coli*, *E. nana*, *D. fragilis*, *G. lamblia*, *Salmonella* spp.

Precision


Table 4: Intra-assay (within-run) precision of the CoproELISA™ *Entamoeba* test is shown below:

Sample	No. of Replicates	Mean value	CV%
Positive	10	1.617	4.7
Negative	10	0.040	4.3

Bibliography

1. WHO (1997) News and activities. *Entamoeba* taxonomy. Bull World Health Organ 75: 291–293.
2. Diamond LS and Clar. A redescription of *Entamoeba histolytica* Schaudinn, 1903 (Emended Walker, 1911) separating it from *Entamoeba dispar* Brumpt, 1925. k CG. A redescription of *Entamoeba histolytica* Schaudinn, 1903 (Emended Walker, 1911) separating it from *Entamoeba dispar* Brumpt, 1925. J Eukaryot Microbiol. 40(3): 340-4 (1993).
3. Tanyuksel M and Petri WA., Laboratory Diagnosis of Amebiasis. Jr. Clin Microbiol Rev. 2003 October; 16(4): 713–729.
4. Fotedar R, Stark D, Beebe N, Marriott D, Ellis J, Harkness J. Laboratory diagnostic techniques for *Entamoeba* species. Clin Microbiol Rev. 2007 Jul;20(3):511-32.

CE

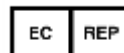

European Authorized Representative: Obelis s.a.
Boulevard Général Wahis 53, B-1030 Brussels, Belgium
Tel: +32.2.732.59.54 Fax: +32.2.732.60.03
E-mail: mail@obelis.net

	Temperature Limitation
	Consult instructions for use
	In Vitro Diagnostic Medical Device
	Manufacturer
	Authorized European Representative