



Vaginal Yeast Test

A rapid, easy-to-use, at-home test for the detection of vaginal yeast (*Candida* species) in vaginal discharge.

Please read all instructions before performing the test.

When to use the test:

You should use this test if you have at least one of the following symptoms:


- Acute itching
- Vaginal discharge, soreness, irritation
- Rash on outer lips of the vagina
- Genital burning that may increase during urination



Directions for use:

- Do not open this test kit until you are ready to use it.
 - Do not use test if pouch is damaged.
 - Make sure that you have a watch available.
 - Make sure to perform the test on a clean surface, wash your hands!
1. Remove the device from the sealed pouch and place it on a flat horizontal surface.
 2. Hold the device stable with one hand. Carefully remove the aluminum cover of the purple cap with the other hand. **Note! The purple cap contains liquid.**
 3. Remove swab from the wrapper.
 4. Insert the swab one inch (2cm) inside the vagina and rotate **for 20sec.** Pull the swab out carefully!
 5. Rotate the swab in the liquid placed in the purple cap, **for 20sec.**
 6. Pull out the swab carefully while squeezing it against the inner wall of the purple cap. **Discard the swab.**
 7. Hold the device stable with one hand. **Turn the purple cap counterclockwise until it stops (b), and then turn it back to starting point (a).**

8. Repeat step no.7 two more times. **In the final position the notches on the purple cap and the white base must be aligned (b).**

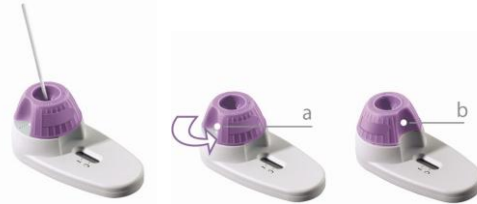

9. **Read results after 10 minutes.** A blue line should appear at the control (C) mark. In case of positive results, an additional blue line should appear at the test (T) mark. **In case of negative or unclear result read again after 10 more minutes.**

Interpretation of results:

1. **Positive** - Appearance of both lines-Detection of yeast infection.



NOTE: The intensity of the color in the test line region (T) should be considered positive even if the colored line is faint.

2. **Negative** - Appearance of only the control line (C). No detection of yeast infection.



NOTE: If both lines do NOT appear or only the TEST LINE (T) appears, no results can be derived from the test. You must perform a new test, while adhering strictly to the instructions.

Test limitations:

- The Vaginal Yeast Test is not reusable.
- The test works only if the instructions are followed carefully.
- Negative test results do not eliminate other infections. Consult your physician!
- The test should not be used during the menstrual period.
- Do not use this test within 7 days after taking medications for vaginal infections.
- Do not use this test within 24 hours after using cream, gel, foam, douching solutions or any other vaginal product.

Note:

- Do not use after the expiry date stamped on the foil pouch.
- Store in a dry place at room temperature between 2^o-30^oC / 36^o-86^oF.
- Keep out of reach of children.
- For in-vitro diagnostic use only. Not to be taken internally.
- Dispose of the device and the swab in the regular trash after use.

Questions & Answers:

Q: What should I do if the test result indicates a negative result?

A: You should consult your physician regarding other infections.

Q: What do I do if the test result indicates a positive result?

A: Please consult you physician for vaginal yeast infection treatment.

Q: In case of a positive result, will both lines be equal in strength?

A: Not necessarily. It does not matter which line is darker. Even a faint line indicates a positive result.

Q: How important is the time frame for reading the results?

A: It is important to read the results between 10-20 minutes but not after 20 minutes.

Q: How do I know if I used the swab correctly and obtained enough sample?

A: The test is sensitive enough to detect a small amount of the yeast.

Q: Can I use the test if I have recently used vaginal cream, gel, foam, or douching solution?

A: Do not use the test if you have applied one of the above materials within the recent 24 hours.

Q: Can I use the test after taking vaginal medication?

A: The test might be affected by various vaginal medications, it is not recommended to use the test within 7 days after taking medications for vaginal infections.

Q: What if no line appears in the window?

A: The test is not valid.

Q: What if the device drops?

A: If all components remain intact the test is usable. However, if any of the components, sealing, etc. is damaged, then the test is not valid.

Q: Is there a preferred time of the day to perform the test?

A: The test may be performed at any time of the day.

Q: What if part of the fluid in the cap of the device is spilled out during or after opening of its cover?






A: In any of these cases, the test is not valid. The amount of the fluid in the cap is crucial for the correct function of the test.

Q: What if I dropped the swab before performing the test?

A: Do not use the test.

Q: If I rotated the cap by mistake before opening the seal or introducing the swab into the cap, can I return it to the initial position in order to perform the test?

A: No. Once the upper part of the device is rotated it is not reversible and the test is not valid.

| | |
|---|------------------------------------|
|  | Temperature Limitation |
|  | Consult instructions for use |
|  | In Vitro Diagnostic Medical Device |
|  | Manufacturer |
|  | Authorized European Representative |



Savyon Diagnostics Ltd.

3 Habosem St. Ashdod 77610, ISRAEL

Tel: +972-8-8562920

Fax: +972-8-8523176

E-mail: info@savyondiagnosics.com



European Authorized Representative: Obelis s.a.

Boulevard Général Wahis 53, B-1030 Brussels

Tel: +32.2.732.59.54 Fax: +32.2.732.60.03

E-mail: mail@obelis.net Mobile: +32.475.45.46.60

If you have any questions regarding the instructions or your results, please call our professional staff at: +972-8-8562920

